



Superarm 129 Basement Model

Making life accessible since 1959

Safest lift in the industry*

SAFE
USER-FRIENDLY
PURCHASED BY THE V.A.
ELECTRIC
RELIABLE
AFFORDABLE
RESALE VALUE RETAINED
MAKES PARKING EASY



Your Coach or Motorhome is designed for your safety, relaxation & enjoyment. Your mobility equipment should be designed with the same principle in mind. The last things you want are high cost & spending your time fighting your equipment. The #129 Basement Motorhome Lift is the safest (per NHTSA studies) & most unobtrusive lift you can own. It is designed for Motor Coaches or mobile homes with a ground to floor height greater than 34 inches, & a door opening of at least 3" wider than the wheelchair. Only 9 inches of mounting space is required on the inside wall by the door for the body of the lift. (If ground to floor is less than 34 inches see #124 Motorhome lift).

| Features | Benefits |
|---|---|
| · Safest Lift available (per NHTSA studies). | Peace of mind in knowing you & your loved ones are safe & secure with no platform to fall off of. |
| · Compact lift body 9" wide, Arm 3 ½ " wide | · Doesn't block windows. · Preserves you precious floor space. |
| · No Platform | · Leaves doorway open. · There is no platform to fall off of. |
| · Minimal modification to vehicle Adaptable for ground to floor from 34"to -51" | · Protect resale value of your vehicle · Maintain transferability of you lift. |
| · All Electric-Low Amperage · Runs off vehicle battery | · Less drain on your electrical system · No extra battery needed |
| · Lift is rated at 600 lbs | · Will lift most chairs; manual, electric, sport style & scooters |
| · Exceeds V.A. Standards | · Financial aid available through some organizations. |
| · Less parking space needed Don't need space for platform to unfold, land & then have space to let you off the end. | · Requires loading space only the size of the wheelchair |
| · Convert vehicle to fit your needs, not the lift's. | · Less money for purchase |
| · Designed - by the disabled | · Unique perspective of the disabled community travelers' needs. |
| · No hydraulics | · Dependable in all weather No leaks or freeze ups |
| · 3 year warranty (5 year limited) | · 40+ years of service & standing behind our products |
| · Standard manual crank down on the motor uses 1/2-inch ratchet wrench. | · Back up feature for manual operation. |
| · Lift fits all year makes & models of Coaches or Motorhomes with 34-51 inch ground to floor height & door opening 3 ½ inches wider than chair. | · If you purchase a new Coach or Motorhome you can transfer the lift from one Motorhome to another without buying a new lift. Your investment is protected. |



One of the biggest drawbacks to platform Motorhome lifts, is that they require a costly separate entrance taking up valuable floor space. With platform style lifts even if they use the existing doorway, the platform is always in the way, blocking the windows & other people from getting in & out. With the Superarm Lift series, no doorways are blocked, no platform is used & less parking space is needed because, you don't require space to fold out a platform & then have room to wheel off the end. It sets you down right beside the Motorhome. The Superarm series of lifts are also the quietest lifts made. The installation of this lift is easy & fast. A qualified installer can install it in approximately four to six hours. The lift is powered by the Motorhome's electrical system, so no extra

battery is needed. It has a low amp ½ hp motor that drives the gears, which allows the arm to pivot in & out of the vehicle. It also has a linear actuator that controls the up/down of the lift unit. This allows for the additional height clearance needed for your chair to land on the floor of the vehicle. Each lift comes with 1-inch wide mountain climber type webbing straps. The straps are inconspicuous & comfortable. This webbing material has been tested at over 1700 lbs. You are fully secured while loading & unloading. (Lift capacity

600 lbs.) The switches for the operation of #129 Basement Motorhome Lift are located on a hand held pendant with switches attached to the lift via a cord. The cord length is sufficient so that the user can easily load & unload themselves, or an attendant can operate the switch pendant. There are two toggle switches on this pendant. One is to operate the in out function of the arm, the second switch controls the elevation of the lift unit. Many users feel that the Superarm lift has completely revolutionized the way they approach mobility & their vehicle.



- Available door entry must be at least 3 ½ inches wider than your wheelchair at its widest point. Ground to floor height of 34"-51"
- Lift requires 9 inches of mount space on the wall inside the door on either the left or right hand side. (Lift is not interchangeable, but be ordered Left or Right)
- Most chairs will work, however chairs cannot exceed 39 ½ inches long.
- If you have a scooter, please let us know the type & length & we will let you know the requirements.
- Child Inserts, or oxygen tanks on the back of the chair may cause webbing balance challenges. Please let us know the specific type of chair & Motor home measurements & we will help you with the requirements of the Superarm.
- Step well must have some covering built over it allowing a safe landing area on